

Using the MedRx REM Plus Probe Microphones with the AVANT REM Speech



The REM Plus Probe Microphone is compatible with the AVANT REM Speech system, however, the indicator lights will not illuminate.

Operation of the REM Plus Probe Microphones is identical to the original AVANT REM Speech Probe Microphones with the following exceptions:



The Calibration nubs are located on the clear plastic Reference Microphone cover shown here.



The Probe Microphone assembly hangs on the ear as shown. Note that the indicator light will not illuminate.

Note: To adjust location of Probe Microphone, pull on the black loop.